



IVD



3diag - B2m - 800

REF R-521

LOT **521-40**

2026-03

EN Beta-2 Microglobulin (serum+urine) - for *IMMAGE*[®] 800

INSTRUCTIONS FOR USE

ES Beta-2 Microglobulina (suero+orina) - para *IMMAGE*[®] 800

INSTRUCCIONES DE USO

IT Beta-2 Microglobulina (siero+urina) - per *IMMAGE*[®] 800

ISTRUZIONI PER L'USO

PT Beta-2 Microglobulina (soro+urina) - para *IMMAGE*[®] 800

INSTRUÇÕES DE UTILIZAÇÃO





<https://www.3diag.com/002>

for IFU, Scan or follow link,
and select LOT

IMMAGE[®] is a registered trademark of Beckman Coulter, Inc., Fullerton, CA

CATALOGUE of Available Proteins - Turbidimetry (TIA) / Nephelometry (NIA)

Protein Description	Traced to	Sample	Beckman C. IMAGE®	Binding Site Optilite®	Siemens H. BN™/Atellica®	Turbidimetry see NOTE #1
 Free Light Chains Kappa - Serum	ERM-DA470k	Serum	TD-42500-K	-	-	TD-42510-K
 Free Light Chains Lambda - Serum	ERM-DA470k	Serum	TD-42500-L	-	-	TD-42510-L
 Free Light Chains - Urine	ERM-DA470k	Urine	TD-42500	-	TD-42505	TD-42510-U
Kappa (B&F) Light Chains	ERM-DA470k	Serum & Urine	-	TD-42777 (serum) TD-42787 (urine)	-	TD-42775 (serum) TD-42775-U (urine)
Lambda (B&F) Light Chains	ERM-DA470k	Serum & Urine	-	TD-42778 (serum) TD-42788 (urine)	-	TD-42795 (serum) TD-42795-U (urine)
Beta-2 Microglobulin	WHO B2M	Serum & Urine	TD-42520 (sr+ur)	-	-	TD-42535 (sr+ur) TD-42531 (urine)
C1q Complement	WHO W1032	Serum	TD-42540	TD-42547	TD-42550	TD-42555
C5 Complement	WHO W1032	Serum	TD-42560	TD-42567	TD-42570	TD-42575
C1 (Esterase) Inhibitor	Internal Standard	Serum	TD-42580	-	-	TD-42595
Factor B (C3 Proactivator)	WHO W1032	Serum	-	TD-42717	TD-42720	TD-42725
Cystatin C	ERM-DA471	Serum & Urine	TD-42600 (sr+ur)	-	-	TD-42615 (serum) TD-42615-U (urine)
Hemopexin	NIBSC 74/520	Serum	TD-42620	-	-	TD-42635
IgD Immunoglobulins	NIBSC 67/037	Serum	TD-42640	-	TD-42650	TD-42655
Retinol Binding Protein	Internal Standard	Serum & Urine	TD-42660 (sr+ur)	TD-42667 (serum) TD-42677 (urine)	-	TD-42675 (serum) TD-42675-U (urine)
Soluble Transferrin Receptor	WHO 07/202	Serum	TD-42680	TD-42687	-	TD-42694
Alpha-1 Microglobulin	Internal Standard	Urine	-	-	-	TD-42835
Serum Amyloid A	WHO 92/680	Serum	TD-42880	TD-42887	-	TD-42895

NOTE #1: Applications available for **Alinity c**, **Architect c**, **ADVIA®** series, **AU®** series, **cobas®** series, and other analyzers by request

IMAGE® and **AU®** are registered trademarks of Beckman Coulter, Inc, Fullerton, CA.
BN™ and **ADVIA®** are registered trademarks of Siemens Healthcare Diagnostics Products GmbH, Marburg, Deutschland.
Optilite® is a registered trademark of The Binding Site Group Ltd, Birmingham, U.K.
Alinity and **Architect** related brand marks are registered trademarks of Abbott Laboratories, Abbott Park, Illinois, USA
cobas® and related brand marks are trademarks of Roche Diagnostics Ltd, Roetkreuz, Switzerland

CONTROLS INCLUDED IN  **Accuracy 365** Online Quality Control Further information and registration at www.accuracy365.com

Accuracy 365 aims to monitor and compare, between laboratories, the results obtained in the Internal Quality Control processes of end-users of specific protein control materials manufactured by **TRIMERO Diagnostics**.

Main features:

- 100 % free and lifetime.
- Cloud application, Available 24 hours, 365 days per year.
- Intuitive interface and very simple to use.
- Calculation and representation of statistical data in real time.
- Anonymous participation, data are only accessible by its owner.